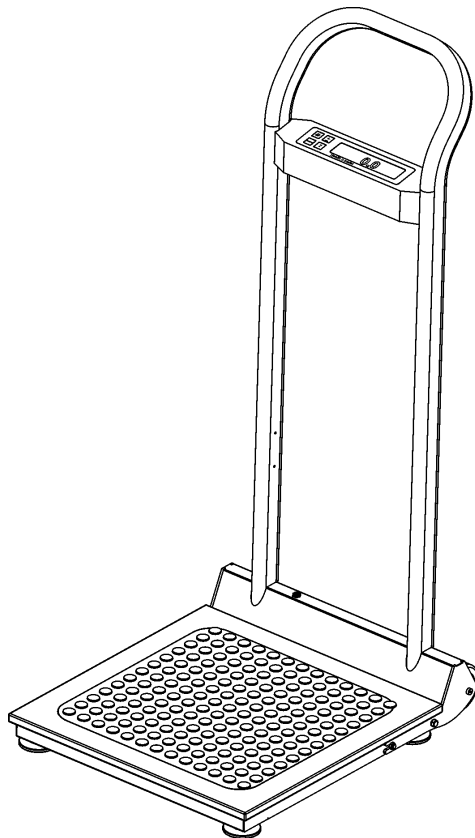
Health o meter®

**HIGH CAPACITY STAND-ON SCALE
MODEL 1101KL**

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OPERATION MANUAL

MODE D'EMPLOI

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HIGH CAPACITY STAND-ON SCALE MODEL 1101KL

Thank you for your purchase of this product.

Please read this manual carefully and keep it handy for ready reference.

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TABLE OF CONTENTS

CAUTION AND WARNING.....	2
SPECIFICATIONS.....	2
ASSEMBLY INSTRUCTIONS	3
SET UP.....	4
OPERATION INSTRUCTIONS.....	6
MAINTENANCE.....	6
TROUBLESHOOTING.....	7
CALIBRATION PATH	8
EXPLODED VIEW OF SCALE	9
PARTS LIST	10
WARRANTY	11

CAUTION AND WARNING

- To prevent injury, do not transport the scale while the platform is loaded.
- For accurate weighing the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.
- Operating at other voltages and frequencies than specified could damage the equipment.
- If the “LO BAT” indicator activates, for accurate weighing, replace the batteries or connect the scale to an AC power source as soon as possible.

SPECIFICATIONS

GENERAL

Health o meter’s High Capacity Stand-On Scale Model 1101KL uses highly sophisticated microprocessor technology. Each precision instrument is designed to provide accurate, reliable and repeatable weight measurements and features that make the weighing process simple, fast and convenient.

The actual weighing is performed by motion-sensing weighing technology, to determine the weight of an active human.

The weight can be display in pounds or in kilograms. The scale includes two wheels and a pillar, used to easily mobilize the scale from place to place.

The unit can be operated using its AC power adapter or by 6-C cell batteries.

SCALE SPECIFICATIONS

Capacity and Resolution	700 Lb x 0.2 Lb (310 Kg x 0.1 Kg)	
Power Requirements	Adapter 120V AC - 60Hz 6 x C size batteries	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 30°F to 125°F (0°C to 50°C) Humidity: 85%	
Physical Dimensions	Platform Length: 19 ¹ / ₈ " (48.5 cm) Width: 19 ¹ / ₈ " (48.5 cm) Height: 2 ¹ / ₄ " (6 cm)	Platform with Handrail Length: 23 ³ / ₄ " (60.5 cm) Width: 19 ¹ / ₈ " (48.5 cm) Height: 44 ⁷ / ₈ " (114 cm) Weight: 37Lb (16.8Kg)

ASSEMBLY INSTRUCTIONS

Parts List:

- (1) Platform Assembly
- (1) Handrail Assembly
- (4) Rubber Feet
- (1) AC Adapter Holder

This 1101KL scale has been calibrated at the factory. Remove each assembly from its carton and unwrap packing material carefully to prevent scratching the unit's parts.

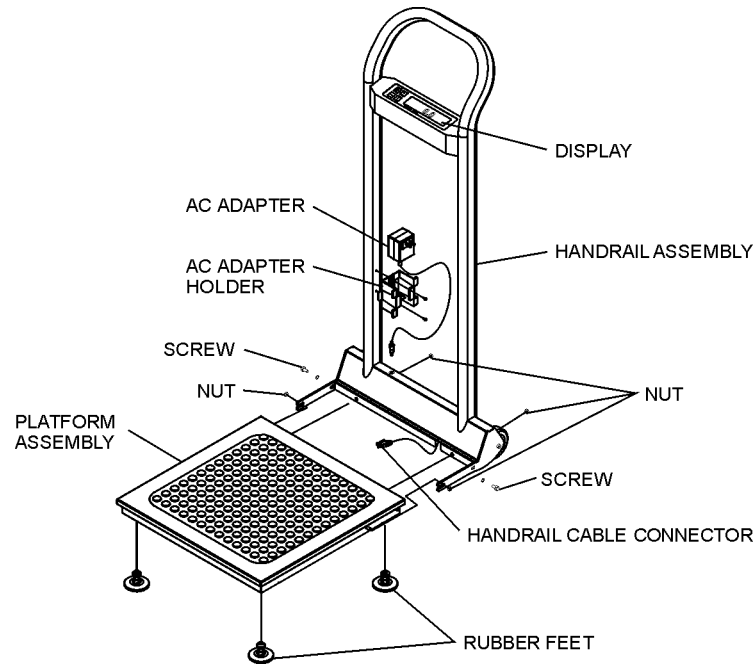
Figure 1. Assembly After Shipment

1. Place the platform assembly upside down on a level floor.
2. Screw the four feet onto the platform assembly.
3. Prepare the platform assembly for installing to the handrail assembly as follows (use 7/16" or 11mm wrench):
 - Unscrew 2 screws from the both sides of the platform assembly.
 - Unscrew 4 nuts from the both sides and from the back of the platform assembly.
 - Keep these screws and nuts for reinstalling in the same locations.
4. Hold the handrail assembly perpendicular to the platform assembly, as shown in Figure 1.
5. Attach the handrail assembly to the platform assembly, install 2 screws at both sides of the platform and then 4 nuts at both sides and at the back of the platform.
6. Plug the handrail cable connector into the platform assembly.
7. Install the AC adapter holder on the interior right side of the handrail using two screws attached to the holder assembly.
8. Adjust the rubber feet for equal stabilization of the scale.

The scale may be operated by an AC power source adapter or by 6-C cell batteries. The scale will automatically switch to battery operation when an AC power source is absent.

SET UP

1. Plug the scale's AC adapter into the handrail assembly and into the power source or place batteries in the battery holder (see "Replacing Batteries" below).
2. Press the ON/OFF button to turn the scale on. The display will show "start" and then "0.0".
3. When the self-test function is complete:
 - The digital display should read "0.0".
 - "LB" should be indicated on the right side of the display. (To change the default units of the scale to kilograms (Kg), please contact our Technical Support Center at 1.800.638.3722 or 1.708.598.9100).
4. Place a weight [not to exceed 700 Lb (310 Kg)] on the scale. The digital display should blink "-----" until the weight calculation is complete.
5. Press the LBS/KG button to select the weight mode, either in kilograms or pounds.
6. Press the REWEIGH button; the digital display should read again the weight of the item on the scale.
7. Remove the weight from the scale. The digital display should continue to read the weight of the item that was on the scale.
8. Press the ZERO button; the digital display should return to zero.
9. Disconnect the power source connector from the scale, then turn the scale on using ON/OFF button. The scale should automatically switch to battery power.

NOTE: If the set up procedure failed, go to the troubleshooting instructions. If the problem is not corrected, refer to qualified service personnel.

Replacing Batteries

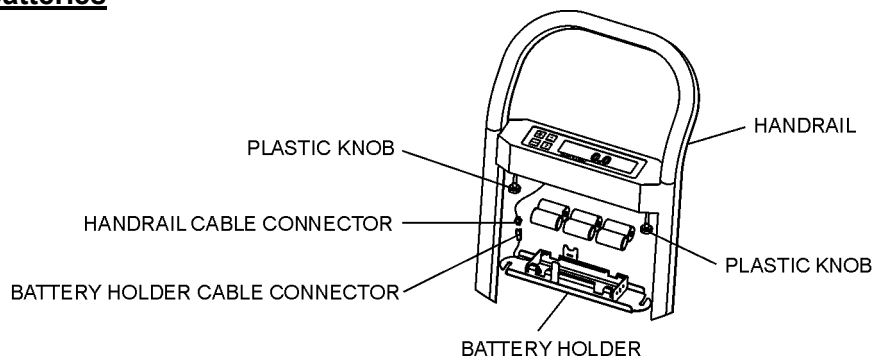
Figure 2. Replacing Batteries

1. Disconnect the scale from the AC power source.
2. Loosen two plastic knobs found underneath the display until the battery holder will be released from the handrail assembly and will rest on the knobs. Carefully pull out the cable to provide access to the connector. Disconnect battery holder cable connector from the handrail cable connector by pressing the plastic button.
3. Raise the battery holder and simultaneously rotate it counterclockwise 1/4 turn. Carefully remove the holder from the handrail assembly.
4. Replace the batteries with new ones.
5. Place the battery holder on the plastic knobs by rotating it 1/4 turn clockwise.
6. Plug the battery holder cable connector to the handrail cable connector.
7. Hold the battery holder close to the handrail assembly and tighten the two plastic knobs to secure in place. Be careful not to damage the cable.

SET UP

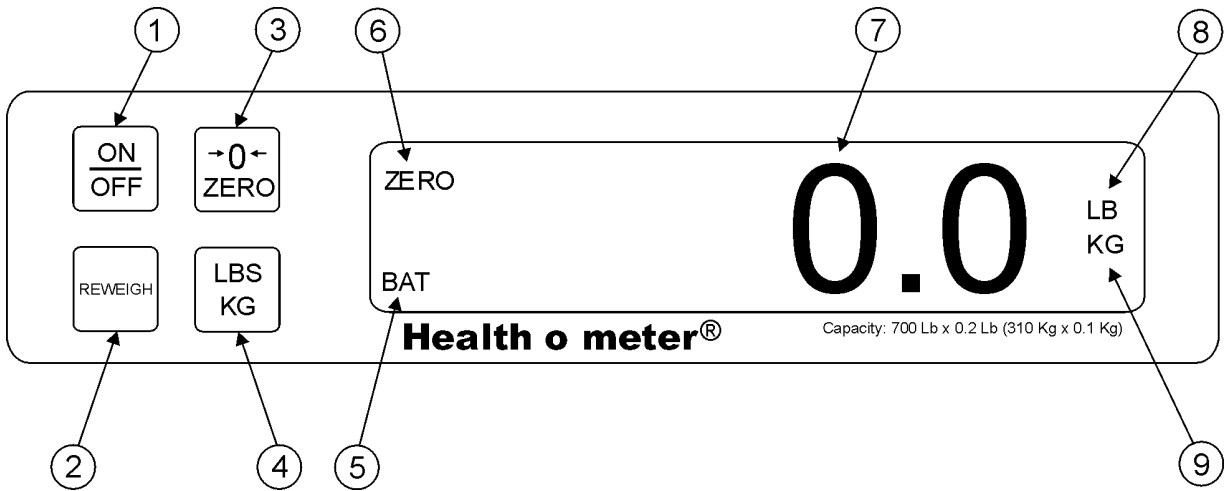
Figure 3. Control Panel

ITEM	DESCRIPTION	FUNCTION
1	ON/OFF button	Turns scale ON and OFF.
2	REWEIGH button	Allows repeated reweighing of person.
3	ZERO button	Zeros Digital Display prior to weighing. Enables the cancellation (tare) of weight if some object was placed on the scale.
4	LBS/KG button	Selects kilograms or pounds.
5	BAT indicator	When ON, indicates scale batteries are low. The batteries should be replaced.
6	ZERO indicator	When ON, indicates the scale is stable and ready for use.
7	Digital Display 6 Digit LCD	Displays human weight in kilograms or pounds as shown by the "LB" or "KG" indicator on the right side of the display. When REWEIGH button is depressed, display will blink "-----" until patient's weight is determined.
8	LB indicator	When ON, indicates displaying weight in pounds.
9	KG indicator	When ON, indicates displaying weight in kilograms.

OPERATION INSTRUCTIONS

1. Press the ON/OFF button to turn the scale on.
2. Wait until "0.0" appears on the digital display.
3. Ask the patient to step on the scale. The display will blink "-----" until the weight of the person is determined.
4. If you wish to reweigh without asking the patient to step off and to step on the scale again, press the REWEIGH button.
5. Ask the patient to step off the scale. The digital display should continue to read the person's weight until REWEIGH or ZERO button is pressed.

NOTE: After 2 minutes of inactivity, the scale will turn itself off to conserve power. To restart the scale press the ON/OFF button.

MAINTENANCE

GENERAL

This section provides instructions for maintenance, cleaning, troubleshooting and operator replaceable parts for the High Capacity Stand-On Scale Model 1101KL. Maintenance operations other than those described in this section should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Inspect AC adapter for cord cracking or fraying or for broken or bent prongs.

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

Disconnect the scale from the AC power source.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale platter or pillar assembly into cleaning or other liquid solution.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.


TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting service personnel.


SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	<ol style="list-style-type: none"> 1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply 	<ol style="list-style-type: none"> 1. Replace batteries 2. Use a different outlet 3. Replace adapter
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> 1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration 	<ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Ask the patient to get off the scale, zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value
The display shows "OL"	The load on the scale exceeds the capacity	Remove the excess weight and use the scale according to its limits
The display shows "UL"	The scale is in under-load condition	Make sure the weighing tray is placed on the base
The display shows "LO BAT"	The battery is low	Replace batteries according to instructions

CALIBRATION PATH

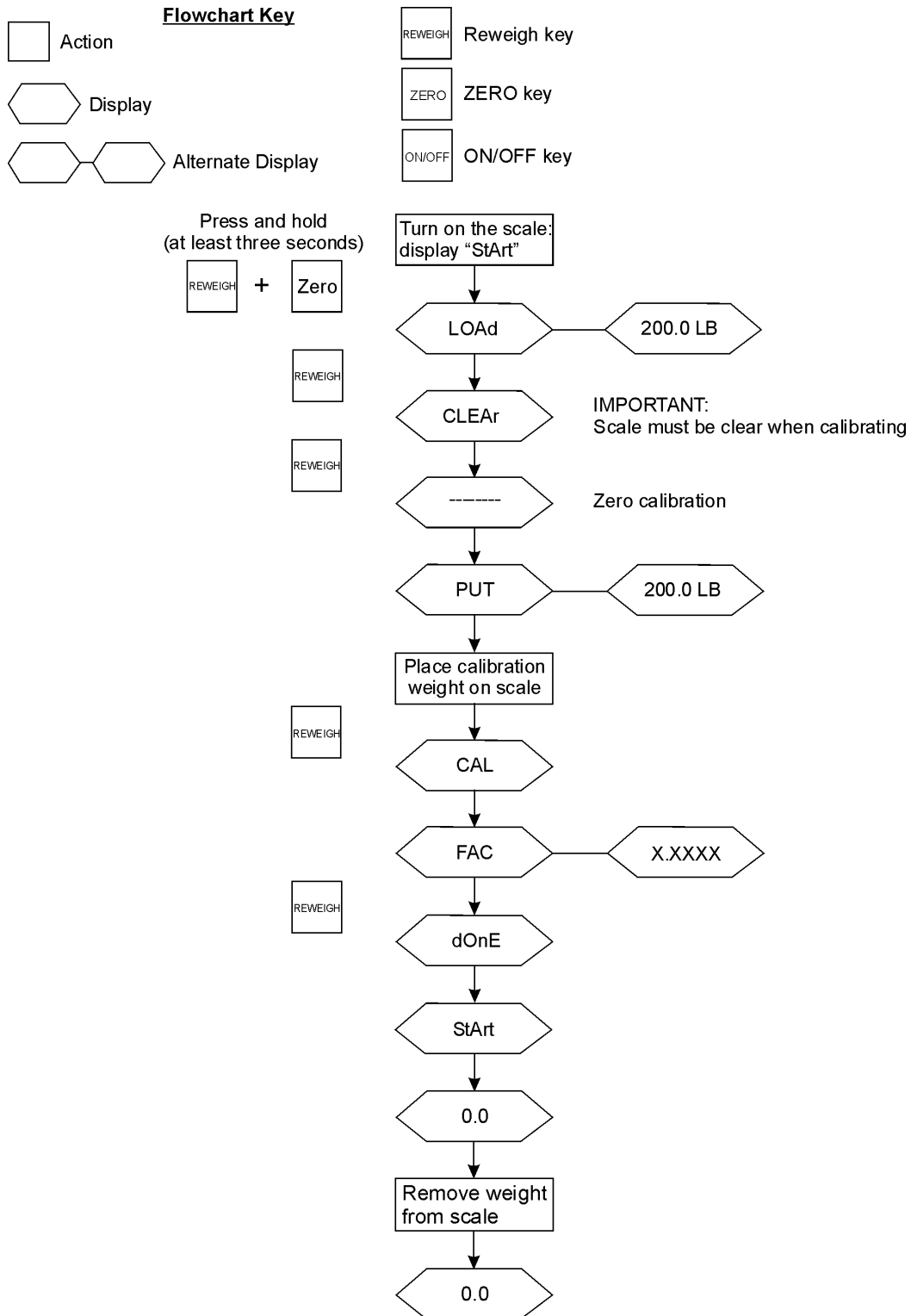The modes are accessible by holding certain keys pressed after power-up.
The keys must be pressed and held for at least three seconds.

EXPLODED VIEW OF SCALE

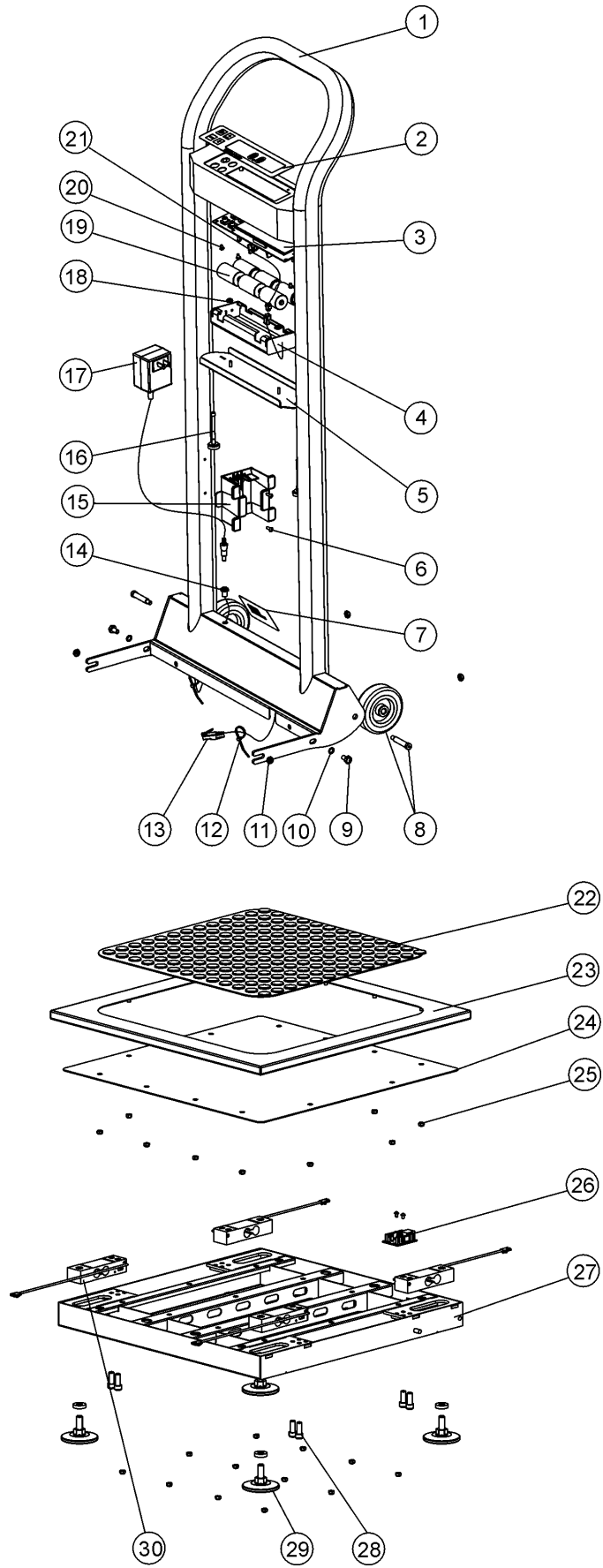PARTS LIST

Key No.	Part No.	Description	Qty.
1	3816501-0	HANDRAIL ASSEMBLY	1
2	3513101-0	KEY PAD	1
3	2138801-0	MAIN BOARD	1
4	2032801-0	BATTERY HOLDER	1
5	3814501-0	DISPLAY HOLDER	1
6		SHEET METAL SCREW #8*1/2"	2
7		MODEL LABEL	1
8	2266601-0	PLASTIC WHEEL ASSEMBLY	2
9		HEX. HEAD SCREW NC1/4*1/2"	2
10		SAFETY WASHER NC1/4	2
11		SELF LOCK NUT NC1/4	4
12		CABLE TIE 10 CM	2
13	2033601-0	MAIN BOARD – JON. BOX CABLE	1
14	2033501-0	MAIN BOARD – CAHRAG PLUG CABLE	1
15	3817001-0	ADAPTER HOLDER	1
16		BINDING SCREW	2
17	420964	ADAPTER UNIT 120VAC 9VDC	1
18		NUT WITH LOCK WASHER NC6	2
19		6-C CELL BATTERIES	6
20		PHIL. PAN HEAD SCREW NC6*1/4"	8
21	2033301-0	MAIN BOARD – CONN. 2 PIN CABLE	1
22	3816701-0	RUBBER MAT	1
23		COVER	1
24		FLAT PLAIN	1
25		SELF LOCK NUT M5	24
26	2138301-0	JUNCTION BOX	1
27		BASE ASM.	1
28		HEX. SOCKET SCREW NC5/16*3/4	8
29	2266701-0	RUBBER FOOT NC3/8" ASSEMBLY	4
30	1304201-0	LOAD CELL	4

WARRANTY

LIMITED WARRANTY

What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar. See "To Get Warranty Service" below for instructions. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service

Make sure you keep your sales receipt or document showing proof of purchase.

Call 1 (800) 815-6615 or 1 (708) 598-9100 to receive a return authorization number. Attach proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar LLC
Attention R/A# _____
Repair Department
7400 W. 100th Place
Bridgeview, IL 60455

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval prior to servicing or replacing.

Pelstar LLC
7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615 or 1-708-598-9100
www.healthometermedical.com

Health o meter® is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

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Health o meter® es una marca registrada de Sunbeam Products Inc., Boca Raton, FL 33431