
HYUNDAI

Instruction Manual

Automatic Arm

CE FDA
0434 KO14141

Blood Pressure/Pulse Monitor
- Two-Way Measurement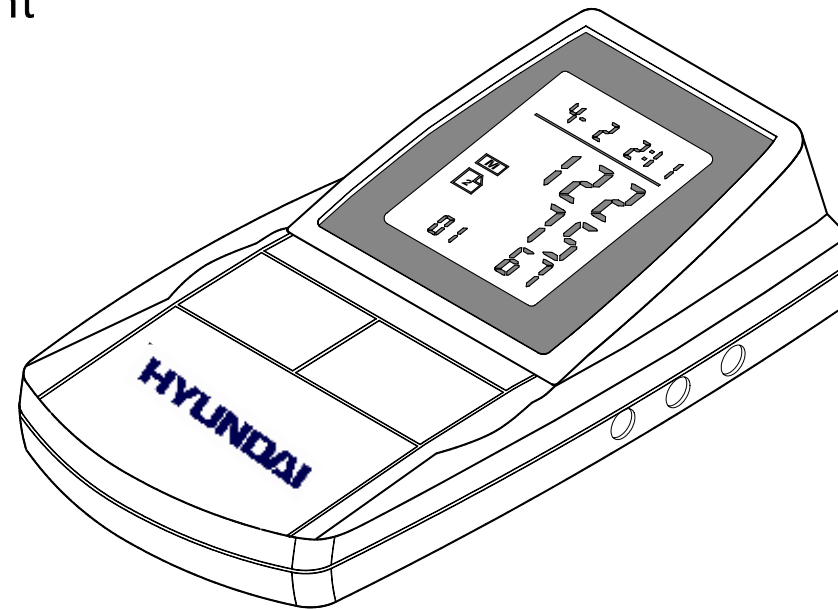

Model: HA211

Table of Contents

Page

Knowing the Functions/Features	1 - 2
Before Taking your Measurements	3
How to Measure your Blood Pressure	4 - 7
How to Set and Date	8
Blood Pressure Classification	9
Information on Blood Pressure	10
Why Monitor at Home	11
Memory	12
Taking Care of your Monitor	13
Troubleshooting	14
Warranty	15
Specifications	16

Hyundai Preventative Care

Thank you for purchasing Hyundai's Automatic Arm Type Blood Pressure Monitor!

Purchasing this product indicates that you are now on your path to a healthier lifestyle through preventative care. We at Hyundai are dedicated in providing quality products so those that practice preventative care can live a longer, healthier life.

Monitoring your blood pressure at home is your first step to a healthier lifestyle but we at Hyundai recommend you consult your doctor regularly, stay active through exercise and eat healthy. Adjusting your diet to include more fruits and vegetables, whole grains, fish and low/non-fat dairy products is a great way to start. Also make sure to check on your sodium intake. As always, consult your doctor before beginning any dietary changes.

Important Warnings/Medical Disclaimer


Prior to using your blood pressure monitor, you should read, understand and follow all instructions and warnings.

Information in this manual is provided for information purposes only. This manual and product are not meant to be a substitute for the advice provided by your physician or any other medical professional. You should not use the information contained herein or this product for diagnosing or treating a health problem, disease or prescribing any medication. Should you suspect or have a medical problem, promptly contact your personal physician, healthcare provider or call 911.

Hyundai blood pressure monitors are recommended for use by people over the age of 18 and are not suitable for clinical use.

KNOWING THE FUNCTIONS


- ① POWER: power on /off
- ② START: start measuring / measure again
- ③ PAGE: press “PAGE” once to choose group (1,2)
- ④ TIME: Time adjustment, press down “TIME” for 5 seconds until the digits of year/ month/day/time starts blinking, press “MEMO” to increase; press “PAGE” to decrease
- ⑤ MEMO: recall different previous values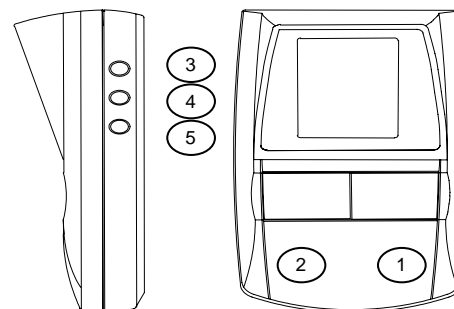


FEATURES OF THE PRODUCT

- Large LCD display with year, month, date and time
- 2-person memory zones
- 30-reading for each zone (60 total memory)
- Auto shut-off in 3 minutes


▪LCD Panel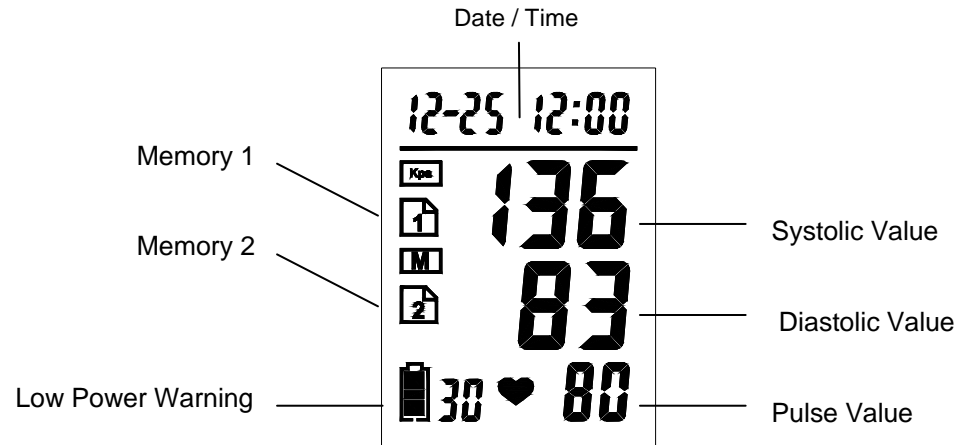


▪Arm Cuff

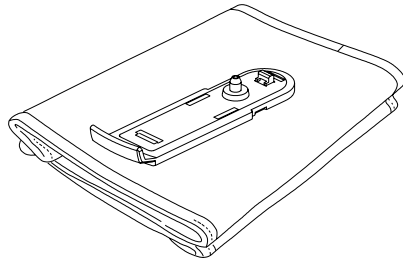▪Bottom of the Main Unit

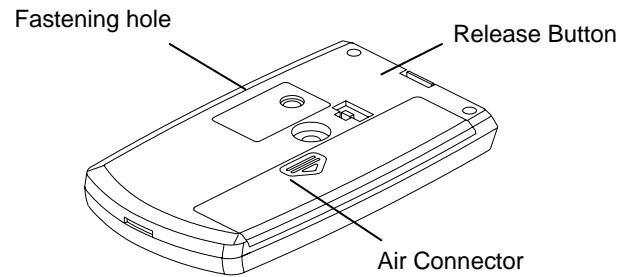▪Air Tube

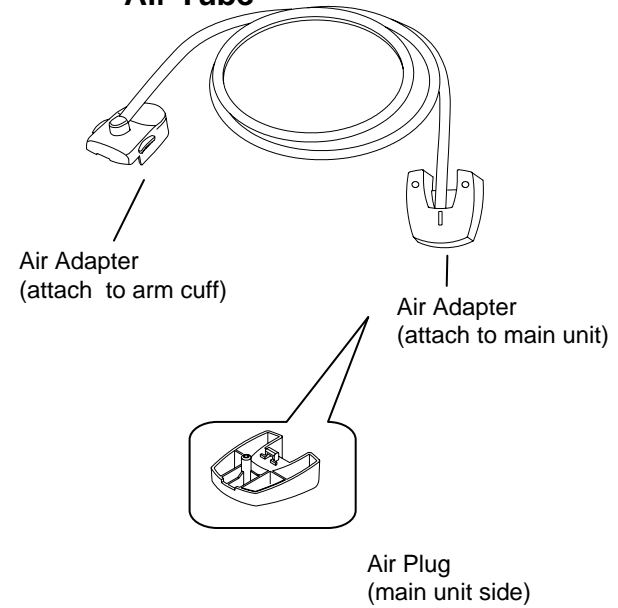Before taking a measurement:

1. Sit quietly for about 10-15 minutes, this will allow your body to return to its normal, resting state.
2. Take your blood pressure in a comfortable environment.
3. If you have consumed any beverages containing caffeine such as coffee, teas, or cola, wait about 30-45 minutes. Do not smoke prior or during taking a measurement.

While taking a measurement:

1. Sit quietly and still while measuring. Talking or moving may lead to elevated measurements.
2. For consistency, it is suggested using the same arm and at about the same time each day. We suggest using your left arm if possible.
3. When taking multiple measurements right after each other, wait at least 10 minutes in between. This allows your blood pressure to return to their normal state.
4. Follow the instructions in this manual on how to measure your blood pressure.

Note: Blood pressure constantly changes during the course of a day. Occasionally, high and low readings do not necessarily indicate your monitor is malfunctioning or that a serious condition exists. It is important to speak with your doctor to learn what your normal blood pressure levels should be at and if you should experience related abnormal measurements. Should you feel you cannot wait to speak with your doctor, call 911.

HOW TO MEASURE BLOOD PRESSURE

1. Insert the batteries

1. Remove the battery cover in accordance with the direction of arrow.
2. Insert 4 x "AAA" battery in accordance with the indication of polarities on the batteries.
3. Slide the battery cover to close.

2. Two Ways Measurement

To attach the main unit with the arm cuff

Fig.1 Connect the air connector with the air plug and securely affix the fastening hole to the fastening protrusion.

To detach the main unit from the arm cuff

Fig.2 Connecting the air adapter of air tube with the air connector of the main unit.

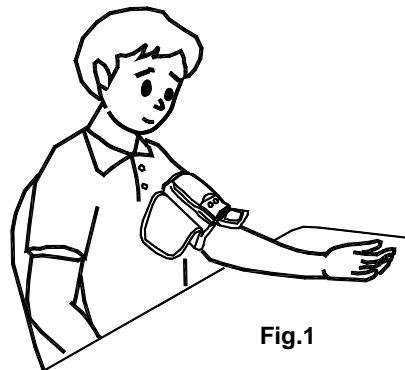
Fig.1

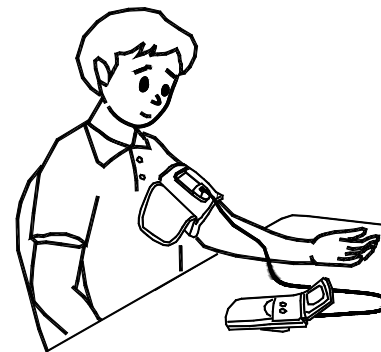
Fig.2

3. To apply the arm cuff.

1. Turn the palm of your hand upward. Place mark cloth inside your arm and match the end of the arm cuff to approximately 1-2 cm from the flexed side of the elbow joint of your arm (chest level) and wrap the arm cuff on the arm. (Please see Figure 3,4)

2. Hold the end of the arm cuff and wrap it up by gently pulling so that there is no gap between the arm cuff and the arm.

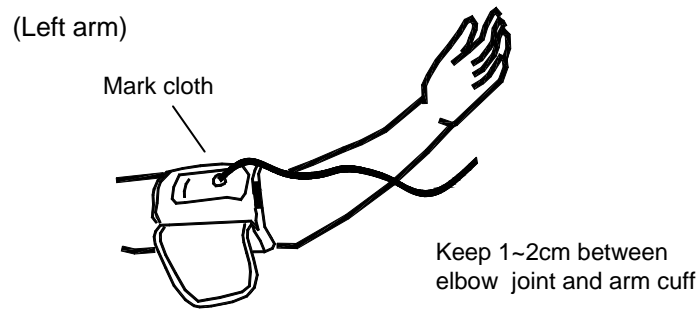
Fig. 3

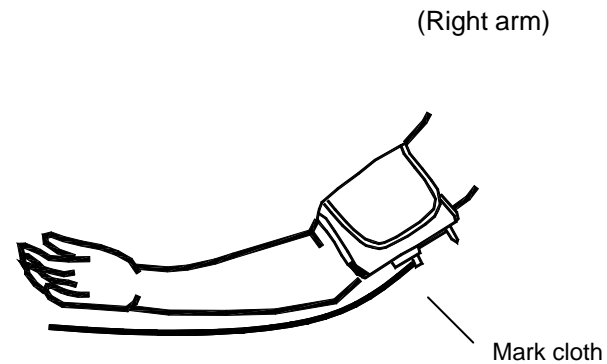
Fig.4

4. To start the measurement

Note: In order to get correct measurement results and to avoid unnecessary erroneous measurements, arm, hand, fingers should be still during measurement and you should not talk. You should be seated.

1. Push the "POWER" button.
2. Push the "START" button after blinking digits stop.
3. Start measuring, the cuff will inflate automatically.

4. Systolic/Diastolic and Pulse Values, all of the measured values will be displayed on the LCD panel.
5. Release cuff from your arm while the cuff deflates completely.
6. To measure 2nd person, after pressing Power and blinking digit stops, press Page, the number 2 will appear, press Start

5. To see the previous measured values.

The main unit automatically stores the previous measured values.

(Even after batteries are replaced, the Previous values are stored in 1 minute.)

1. Press "MEMO" button, you can recall previous measured values for the first person.
2. If you press "MEMO" button successively, you can read next values.
3. Press "PAGE" button, then you can read second person measured values.
4. Stored capacity is up to 30 sets each page.

6. To finish the measurement

1. After deflation, the measurement of blood pressures is completed.

If you want to measure again, just press "START" button again.

Note: We suggest waiting 10 minutes for re-measurements.

2. Press “ POWER” button to switch off electrical supply.
3. If you forget to push the “POWER” button, the power will turn off automatically within 3 minutes.

LOW POWER WARNING

Normally, you can measure your pressure values up to 150~200 times with durable alkaline batteries. If battery icon shows “low power warning” or show only 1 bar, then It’s time to change batteries.

HOW TO SET TIME AND DATE

Generally, LCD panel displays Month/Date/Hour/Minute regarding time message.


If you desire to adjust Year/Month/ Date and Time setting, please follow the procedures below :

1. Hold down "TIME" at least 5 seconds, when Year setting digit blinks, you may begin.
2. Press "MEMO/ >>" to forward "Year" Setting. Press "PAGE /<<"to backward "Year" Setting.
3. After finishing Year Setting, you can press" TIME" button again.

Now, you can adjust Month/Date/Hour and Minute Setting when Month /Date/Hour and Minute digits blink as you desire.

BLOOD PRESSURE CLASSIFICATION

Standards for assessment of high blood pressure, without regard to age, have been established by the World Health Organization (WHO), as shown in the chart as below :

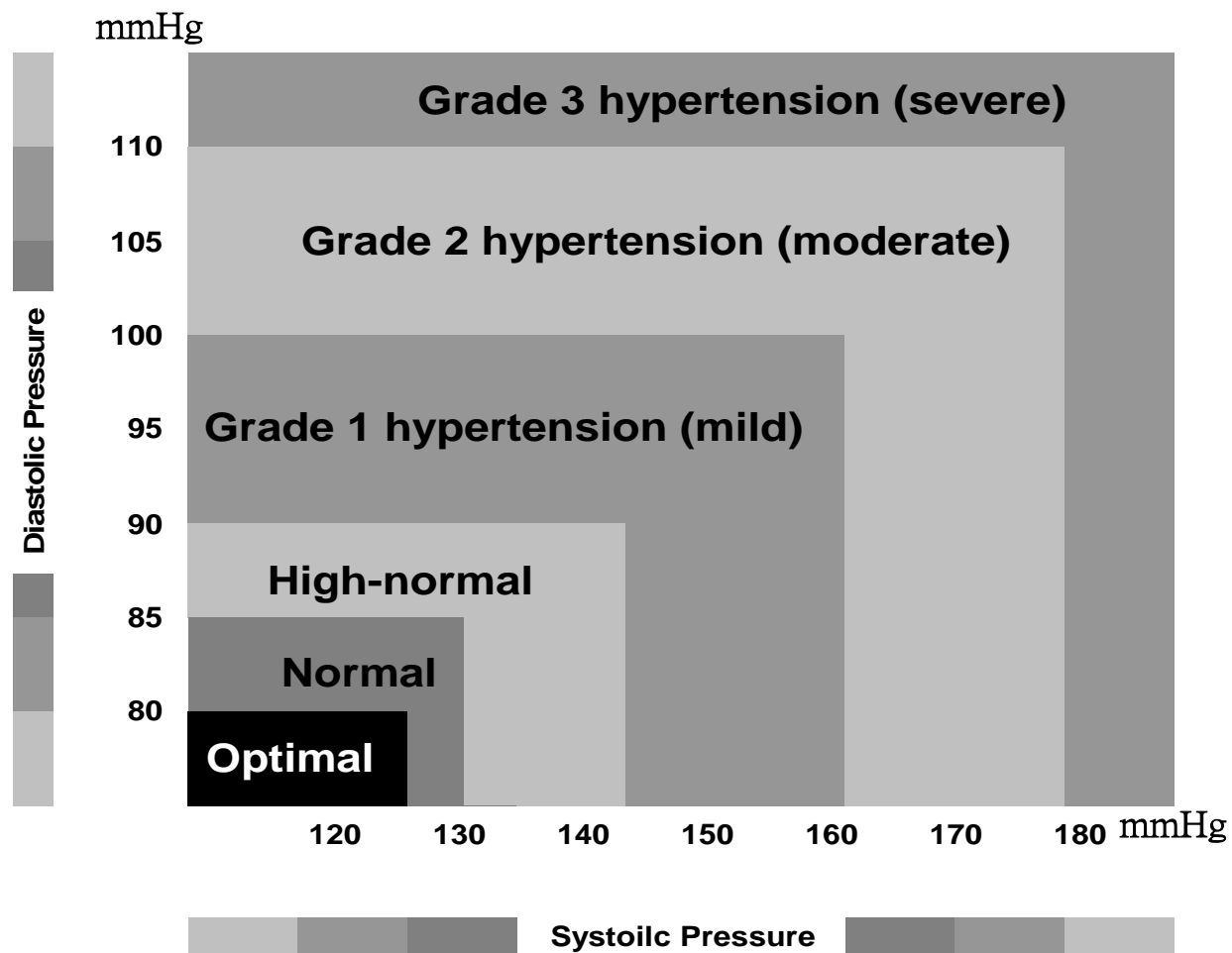INFORMATION OF BLOOD PRESSURE

What is blood pressure?

Blood is sent to the arteries with the action of a pump called heart(contraction and dilation). The pressure of blood leaving the heart is called “blood pressure”. Blood pressures pulsate with each beat of the heart. The high blood pressure when the heart contracts is called “systolic pressure” and the low blood pressure when the heart dilates is called “diastolic pressure”. The threshold value for hypertension in adults is defined by the World Health Organization (WHO) as 140/90 mmHg .

Health and blood pressure

When people reach middle age, the risk of hypertension markedly increase. With aging, the blood vessels age rapidly. Furthermore, because of the obesity and lack of exercise, cholesterols stick to blood vessels, causing them to lose elasticity. Therefore, watching daily blood pressure help evaluate our health condition.

Why do we need to monitor blood pressure at home?

By monitoring at home, you may manage your health between doctor visits. Home monitoring also provides you with a record of blood pressure measurements over time to give your doctor - a valuable tool in hypertensive monitoring. Home monitoring may also alleviate white coat hypertension.

This term refers to the phenomena which causes individuals' blood pressure to be higher when taken in a medical office setting. This is due to the stress of being in a medical office setting which may cause blood pressure to rise higher than normal. When monitoring at home, you may get a more accurate measurement of your blood pressure in a normal, relaxed state.

Why do we need to monitor blood pressure at home?

By monitoring at home, you may manage your health between doctor visits. Home monitoring also provides you with a record of blood pressure measurements over time to give your doctor - a valuable tool in hypertensive monitoring. Home monitoring may also alleviate white coat hypertension. This term refers to the phenomena which causes individuals' blood pressure to be higher when taken in a medical office setting. This is due to the stress of being in a medical office setting which may cause blood pressure to rise higher than normal. When monitoring at home, you may get a more accurate measurement of your blood pressure in a normal, relaxed state.

Sixty Measurement Memory

When you take a measurement, this unit will automatically store in memory. Your unit is capable of storing up to 60 measurements, 30 for each page. To review your measurements and how to store, follow the instructions in the manual.

Taking Care of Your Blood Pressure Monitor

- Keep the unit out of direct sunlight, high humidity and extreme temperature changes.
- Do not drop on the floor
- Do not take the unit apart. Should your unit require repairs, see details in warranty section.
- Keep the unit free of dust and other materials. A bag is provided for safe storage. Avoid getting any parts of the units or cuff wet. Should the unit need cleaning, it is best to wipe it with a cloth slightly dampened with water. Hyundai recommends cleaning your unit once a month.
- Do not inflate the monitor's cuff if it is not wrapped around your wrist.
- For your safety, do not wrapped the cuff around any other body part other than your wrist.

TROUBLES SHOOTING

Symptom	Problem	Solution
Nothing is displayed when you push the Start button.	Are the batteries exhausted?	Replace the batteries with new ones.
	Are the polarities of batteries in the wrong position?	Insert batteries in the correct direction
“Err P” is shown on LCD panel	Fail to inflate	<ol style="list-style-type: none"> 1. Please check if the air adaptor on Main Unit and Cuff are loosen, and connect them. 2. Please check air tube if it is broken and change it. 3. Please check arm cuff If it is broken and change it.
“Err 1” is shown on LCD panel	Deflate too fast	Please contact authorized Hyundai dealer.
“Err 2” is shown on LCD panel	<ol style="list-style-type: none"> 1. Movements occur on the arm 2. Other interference signal around BPM 	<ol style="list-style-type: none"> 1. Keep still after you push the Start button. 2. Please remove interfering sources such as magnetic material or cell phone.
“Err 3” is shown on LCD panel	Incorrect measured result	<ol style="list-style-type: none"> 1. You might wear thick jacket and your BPM fails to detect pressure values. Please take off your jacket and measure again. 2. You might use it incorrectly, such as measure pressure value on child.
<ul style="list-style-type: none"> • If correct measurement is impossible even after checking the above-mentioned points, consult at the store where you have purchased the unit or the nearest Hyundai dealer without touching the internal mechanism. • In some very rare cases, there may be error due to the physical condition of the person. In such cases, please consult your doctor. 		

Manufacturer's 5 Year Limited Warranty

Use of this product, whether arising out of any breach of this warranty or any legal theory, all expressed and implied warranties, including the warranties of the merchantability are limited to the applicable warranty period set forth above.

Manufacturer and Distributor assume no responsibility for injuries, damages or penalties incurred or suffered resulting from the use of this product in an improper manner or location other than that for which it is intended or from failure to heed any warning disclaimer or instruction. Some states do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above exclusions or limitations may not apply to you. This warranty gives your specific legal rights and you may also have other rights which vary from state to state. Should your product become defective during this warranty period, please return the product in the original carton prepaid to:

Hyundai Healthcare USA
3250 Wilshire Blvd
Suite 813
Los Angeles, CA 90010
www.hyundaihealthcare.com

Include \$10.95 for return shipping and insurance (non-refundable)

Include with your monitor a letter stating:

1. Your name and street address
2. Daytime phone number
3. Detailed description of the problem
4. Copy of your dated cash receipt or other proof of purchase showing date

***Note: Product maybe replaced or repaired at our option that is defective in material or workmanship**

SPECIFICATIONS

Name	Automatic Arm Blood Pressure Monitor
Model	HA211
Measurement	Oscillometric
Measurement Range	Pressure:20-280mmHg Pulse rate:40-195beats/ min
Accuracy	Pressure: within ± 3 mmHg Pulse rate:within $\pm 5\%$ of reading (Subject to the temp. humidity listed below)
Inflation	Automatic inflation with pump
Pressure Detection	Semiconductor
Power Supply	4x "AAA" alkaline batteries
Auto Power Off	3 minutes after last key operation
Operating Temperature/ Humidity	+10°C to +40°C, 30% to 85% RH
Storage Temperature Humidity	-20°C to +60°C, 10% to 95% RH
Weight of Main Unit	Approximately: 0.66 lbs. / 300 g (excluding batteries)
External Dimensions	2.59(W) x 4.76 (H) x 0.78(D) inches / 66(W) x 121(H) x 20(D) mm
Cuff Dimensions	5.51(W) x 20.86(H) inches / 140(W) x 530(H) mm (Weight approximately: 0.31 lbs. / 140g) Length of air tube: 27.56 in / 700mm

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