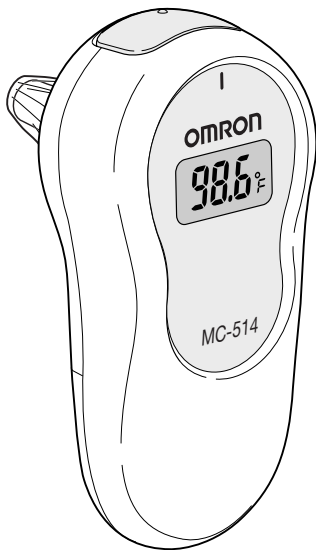
# OMRON®

## INSTRUCTION MANUAL

# EAR THERMOMETER

Model **MC-514**

ESPAÑOL

ENGLISH

# TABLE OF CONTENTS

Introduction .....	3
Part Names .....	4
Conversion °F/°C .....	4
Before Use .....	5
Directions for Use and for Memory .....	5
Care and Maintenance .....	6
Battery Replacement .....	6
Specifications .....	7
Troubleshooting .....	8,9
Warning .....	10
FCC Statement .....	11
Warranty .....	12
Notes .....	13-15
Quick Reference .....	16

# INTRODUCTION

Thank you for purchasing the Omron Ear Thermometer.

<p><i>Fill-in for future reference</i></p> <p>DATE PURCHASED: _____</p> <p>SERIAL NUMBER: _____</p>
---

<p><i>Staple your purchase receipt here.</i></p>
--

This ear thermometer offers a safe, accurate and quick temperature reading from the ear canal. This thermometer senses the infrared heat from the ear drum (tympanic membrane).

No need to worry about broken glass or mercury hazards.

An ear temperature is not affected by talking, eating or drinking. These factors do affect an oral temperature.

In one second it changes the heat measured into an equivalent temperature.

The unit is easy to use for children and adults. Temperatures can be taken when a child is sleeping.

We recommend you practice taking a temperature with the Omron Ear Thermometer on each member of your family when they are healthy. This way you will be confident in your technique when using the ear thermometer on a sick family member.

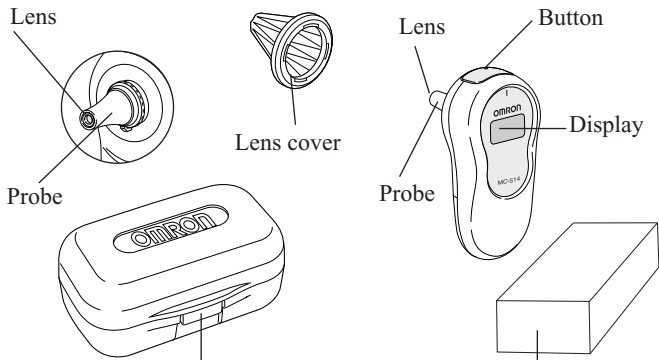
Contact your doctor if you have any questions regarding specific temperatures.

*This product is intended for home use only.*

**Read this instruction manual completely before using your Omron Ear Thermometer.**

## SAVE THESE INSTRUCTIONS

# PART NAMES

Storage case (with 10 lens covers inside, plus 1 on the thermometer)

Box for lens covers (with 10 lens covers inside)

## CONVERSION BETWEEN FAHRENHEIT to CENTIGRADE (°F/°C)

- The thermometer is initially set at Fahrenheit °F
- Follow the steps below to change the measurement scale from °F to Centigrade °C:

### Note:

Changing the measurement scale "°F" to "°C" or "°C" to "°F" deletes the previous measured value.

- 1) Be sure unit is in the off position (blank display).
  - 2) Press and hold the Button for 3 seconds until the "°F" flashes and you hear one beep.
  - 3) Press button again, now "°C" flashes.
  - 4) After five (5) seconds the unit will shut off.
- The desired mode has been set.
  - Repeat the process to reverse measurement scale if desired.

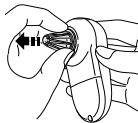
# BEFORE USE

- The thermometer will not work unless a lens cover is applied correctly.
- For hygienic use, attach a new lens cover each time.


## REPLACING LENS COVER

Use Omron lens cover (model MC-505LC).

- 1) Remove the lens cover.
- 2) Attach a new lens cover.

Insert the lens cover until it clicks.
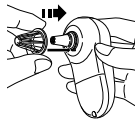
# DIRECTIONS FOR USE AND MEMORY

- (1) Push button to turn on. (The display will illuminate for 2 seconds.)

## Memory Feature:

- (2) The previous value measured is shown on the display (Memory). For the 2nd and following measurements, the previous value measured is shown with “M” symbol (Memory).

**Note:** When the unit is used for the first time the “M” symbol is not displayed.

- (3) When “°F” or “°C” symbol flashes - insert lens into ear. Gently pull back the ear lobe before inserting for best results. Tugging on the earlobe provides a direct line to the eardrum to maximize the accuracy of results.
- (4) Listen for ready beep (1 beep); an “O” symbol will appear indicating its reading. Angle is correct; ready for measurement.
- (5) Press button again while unit is placed in the ear, 4 beeps means measurement is complete.
- (6) Remove probe from ear.
- (7) Discard used lens cover, and replace with a new one, so that unit is ready for use again.
- (8) If you are going to repeat measurements, wait until “°F” or “°C” flashes.

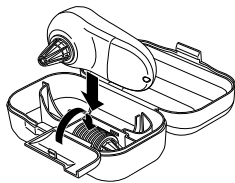
## Note:

- You can measure consecutively up to three times. If you are going to measure more than this, wait ten minutes, then measure again. This is because the measurement may be inaccurate due to the probe warmed by body heat.
- The thermometer will automatically turn off about 1 minute after a temperature has been taken. There is no button to turn the power off manually.

# CARE AND MAINTENANCE

## • How to Clean the Unit

- Wipe off the casing of the unit with a soft, dry cloth, or with rubbing alcohol
- When dirty clean the lens, only use a soft dry cloth or cotton swab.
- Do not clean any part of the unit with detergent, thinner or benzene.
- Do not immerse in water.


## • How to Store your thermometer

- Keep your thermometer in the storage case after attaching a lens cover to protect the lens.
- Store your thermometer in a place that is out of direct sunlight, is not in any extremes of temperature or humidity and is not likely to be affected by vibration, dust or shock.

# BATTERY REPLACEMENT

- (1) Use a lithium battery CR2032.
- (2) Remove the screw and battery cover.
- (3) Remove old battery. Do not use a screw driver or metal tweezers to remove battery.
- (4) Place the new battery with the positive (+) side up.
- (5) Replace the battery cover and tighten the screw.

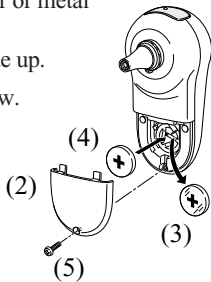**Note:** Dispose of any used battery in accordance with local regulations.

# SPECIFICATIONS

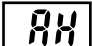
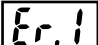
Name:	Omron Ear Thermometer
Model:	MC-514
Power Source:	DC 3 Volt (lithium battery CR2032)
Battery life:	Approximately 2000 measurements
Sensor:	Thermopile
Temperature indication:	In 4 digits, every 0.1°F or °C
Accuracy:	93.2 ~ 96.7°F (34.0 ~ 35.9°C) : ±0.5°F (±0.3°C) 96.8 ~ 102.2°F (36.0 ~ 39.0°C) : ±0.4°F (±0.2°C) 102.3 ~ 108.0°F (39.1 ~ 42.2°C) : ±0.5°F (±0.3°C)
Measurement range:	93.2 to 108°F (34 to 42.2°C)
Weight:	Approx. 1.8 oz (50 g) (including the battery)
Operating temperature / humidity:	Ambient temperature; 50 to 104°F (10 to 40°C) Relative humidity; 30 to 85%
Storage temperature / humidity:	Ambient temperature; -4 to 140°F (-20 to 60°C) Relative humidity; 10 to 95%
Memory:	Memory function is to display the previous reading
Contents:	Thermometer, Storage Case, Instruction Manual, and 21 Lens Covers
Options:	Use Omron model MC-505LC

The specifications may be changed without prior notice.

# TROUBLESHOOTING

Display	Meaning	Action
Nothing is displayed even after unit is turned on.	Dead battery.	Replace battery.
	Polarities of battery (+ and -) are wrong.	Reposition the battery with correct polarities.
“°F” or “°C” flashes, but “0” symbol is NOT displayed	The lens cover may be dirty.	Replace the lens cover with a new one.
	Lens may be dirty.	Clean the lens with a cotton swab.
	Ear drum may be blocked with earwax.	Remove earwax.
 <p>A "0 0" symbol is displayed.</p>	The unit is working under high room temperature, where a special operation is needed.	Place unit in ear, wait 2 seconds. Press button and wait for 4 beeps. Remove from ear, look at results.
 <p>After all the symbols on display illuminate, a "⊗" mark is displayed.</p>	Low battery.	Replace battery.
The display light is not turned on.	Low battery.	Replace battery.

# TROUBLESHOOTING

Display	Meaning	Action
	Lens cover is not on Probe.	Attach the lens cover, then measure.
	Temperature of the unit is unstable.	Make sure unit remains at room temperature where measurement will take place. Allow 30 minutes to stabilize before using.
	The temperature of the unit is 50°F or below which is outside the operating range.	
	The temperature of the unit is 104°F or above which is outside the operating range.	
	May require service.	Turn on the unit again. If "Er.1" is displayed again, the unit may require service. Call Customer Service Dept. 1-800-634-4350
	The measured temperature was above the range of the thermometer.	Probe may not have been properly positioned in ear. The probe must be snug against opening of ear. Be sure lens cover is clean and ear canal is not blocked with earwax.
	The measured temperature was below the range of the thermometer.	

# WARNING

- This thermometer is intended to be used in human ears only.
- Only use the lens cover for this specific unit.
- For proper hygiene DO NOT share lens covers.
- Replace the lens cover when it is effected by ear wax or other body fluids.
- Handle the lens with care. Improper handling may effect temperature accuracy.
- Keep the unit out of the reach of children.
- Use your unit in room temperature. Extremes in environmental temperature may effect the accuracy.
- Store unit at room temperature when possible. Keep thermometer at room temperature for at least 30 minutes before use.
- Remove unit if pain is noted during use.
- DO NOT use if ear infection is present.
- DO NOT use right after swimming or bathing as this may injure ear tissue as well as effect the accuracy of the temperature.
- DO NOT use near a portable phone.
- DO NOT take apart, repair or change any parts of the unit at any time.
- DO NOT get the unit wet or pour any liquid onto any portion of the unit.

# FCC STATEMENT

**Note:**

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)  
This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE  
(for Canada only)

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

# WARRANTY

## Limited 5 Year Warranty

Your Omron® Ear Thermometer (MC-514) excluding the battery is warranted to be free from manufacturing defects for a period of five years under normal use. This warranty extends only to the original retail purchaser.

We will, at our option, repair or replace without charge any necessary parts covered by the above warranties. Repair or replacement is our only responsibility and your only remedy under the above warranties.

To obtain warranty service, ship the monitor and all of the components, together with proof of purchase and a note explaining the problem with \$5.00 for return shipping and insurance to:

Omron Healthcare, Inc.  
Attn: Repair Department  
300 Lakeview Parkway  
Vernon Hills, IL 60061

Be sure to include the model number, your name and your phone number on the enclosed note and any other correspondence.

ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

### FOR CUSTOMER SERVICE

Visit our web site at:

[www.omronhealthcare.com](http://www.omronhealthcare.com)

Call toll free:

1-800-634-4350

# NOTES

# NOTES

# NOTES

## QUICK REFERENCE

- **Press Button**

Pressing the button allows the display to illuminate for 2 seconds, so you can quickly look at the result.

- **Insert Into Ear**

- **Listen For 1 Beep**

- **Press Button Again - Wait For 4 Beeps**

- **Remove From Ear - Read For Results**

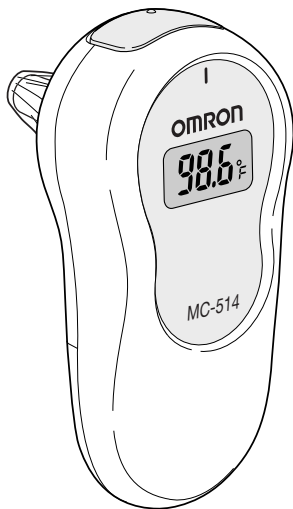
- **Wait 10 Minutes Before 4th Measurement**

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