HT801 Ear-Type Thermometer

Thank you for purchasing Hyundai's Instant Ear Thermometer. The thermometer is designed with infrared technology for fast and accurate temperature measurements. The compact and unique design allows an easy grip handle making it convenient for use at home or travel. Please read the following instructions to achieve the most accurate temperatures and safe operation. With proper care and use, your thermometer will provide you with years of accurate and reliable readings.

Before Use

Before using your new ear thermometer please read and follow the instructions in this manual to insure the most accurate and safe temperature readings. Information herein and product are not meant to be a substitute for the advice provided by your physician.

IMPORTANT SAFETY INFORMATION

1. High, prolonged fever requires immediate medical attention especially for babies and children. Please call your physician immediately or 911.
2. Before you start using this unit, please read this operation manual thoroughly for the correct methods of usage.
3. The thermometer has been designed for practical use, it is not meant to replace a visit to the doctor.
4. To avoid the risk of cross contamination, please use the thermometer individually.
5. Clean the thermometer before and after each use, store in the case provided with the thermometer.
6. The thermometer should only be used under the supervision of an adult. Do not allow children to take their temperatures unattended. Prohibit children from biting, hitting or removing the battery cover, dislodging and potentially swallowing the battery.
7. Do not walk or run during temperature taking.
8. Do not store the unit where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.
9. Do not wash, clean or soak the thermometer with any alcohol.
10. The thermometer must NEVER be sterilized by boiling, gas or steam autoclaves.
11. The use of any other solution or cleaning method beyond the above mentioned may result in reliability problems with the thermometer and void your warranty.

BODY TEMPERATURE

Body temperature can vary during the course of a day and location of temperature reading. Normal body temperature ranges from 97.0°F (36.1°C) to 99.0°F (37.2°C). The generally accepted "normal" temperature is 98.6°F (37.0°C). Usually your temperature is lower in the morning than in the afternoon.

HIGH FEVER ALARM

This thermometer has a built-in fever alarm to warn users of higher than normal temperature. The fever alarm will beep 1 long beep followed by 3 short beeps.

Operating Instructions

1. Gently squeeze the opposite ends of the thermometer to pull off the probe cap. Do not use force in removing the cap.
2. Press the "ON/MEM" button (located below the LCD display).
3. The thermometer is ready for use after the ear icon stops flashing and two beeps are sounded.
4. Gently pull the ear back to straighten the ear canal and snugly position the probe into the ear canal, aiming towards the membrane of the eardrum to obtain an accurate reading (Fig.4-1)
5. Measuring the ear temperature: Use the index finger to hold the "Scan" button (located other side of LCD display at the top) down until you hear a beep (Fig. 4-2).
6. Remove the thermometer from ear, the temperature reading will display on LCD screen.
7. To turn off thermometer, hold down "ON/MEM" button until you see "OFF" on LCD display or device will automatically shut off after 1 minute to extend battery life.
8. Replace probe cap after use.

Figure 4-1

Figure 4-2

 Holding the thermometer too long may cause a higher ambient temperature reading of the probe. This could make the body temperature measurement lower than usual.

Switching between Fahrenheit and Celsius

To change the LCD from °C to °F: When the unit is OFF, hold down the "Scan" button, then press and hold the "ON/MEM" button until the LCD shows "°F", then release the "Scan" button. Use the same process to change the LCD display from °F to °C.

Memory Function

This ear thermometer can store up to 9 readings. The memory function allows you to make temperature comparisons for evaluation of temperature trends.

Using Memory Function

1. If the reading of the thermometer is within the normal temperature range of 93.2 °F to 108.0 °F (34 °C to 42.2 °C), the last measurement data is saved into memory after the thermometer is turned off.
** Press the "ON/MEM" button again to see the temperature stored.
2. To recall memory, press the "ON/MEM" button. The LCD display will show the temperature reading stored in Memory 1.
3. Press "ON/MEM" button again to see the temperature stored in Memory 2. Repeat this process to see temperature measurements stored in all memory locations.

Cleaning and Storage

The Probe is the most delicate part of the thermometer. Use with care when cleaning the probe to avoid damage. Gently clean the exterior and LCD display with a mild solution and do not submerge unit in water or any other liquids. Keep the unit dry and away from direct sunlight.

After each use, please use the alcohol swab or the soft cotton moistened with the 70% alcohol to clean the probe.

Changing the Battery

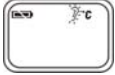
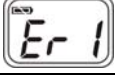
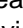
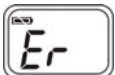
The thermometer is supplied with one lithium cell (CR2032). A low battery symbol will appear on the LCD when the battery needs to be replaced. Power off the thermometer prior to replacing the batteries.

1. Open the battery cover: Insert a pointed object, such as a pen, into battery cover pick hole. At the same time, use thumb to push battery cover out.
2. Hold the device and flip the battery out with a small screwdriver
3. Insert the new battery under the metal hook on the left side and press the right side of the battery down until you hear a click. * The positive (+) side up and the negative (-) side pointed down.
4. Replace the battery cover

The thermometer may be damaged when changing the battery while powered on, turn the thermometer off prior to replacing batteries. Properly dispose of used batteries and keep out of reach of children.

Troubleshooting

Error Message	Problem	Solution
	Device stabilization in process.	Wait until  stops flashing.
	Battery is low and no more measurements are possible.	Replace the battery.
	Measurement taken before device stabilization.	Wait until  stops flashing.
	The device showing a rapid ambient temperature change.	Allow the thermometer to rest in a room for at least 30 minutes at room temperature: 50 °F ~104 °F (10 °C and 40 °C).
	The ambient temperature is not within the range of: 50 °F ~104 °F (10 °C and 40 °C).	Allow the thermometer to rest in a room at least 30 minutes at room temperature: 50 °F ~104 °F (10 °C and 40 °C).
	Error 5~9, the system is not functioning properly.	Remove the battery, wait for 1 minute and reinsert. If the message reappears, contact the retailer for service.
	Temperature taken is higher than 108.0 °F (42.2 °C).	Check the integrity of the probe cover and take a new temperature measurement.
	Temperature taken is lower than 93.2 °F (34 °C).	Make sure the probe cover is clean and take a new temperature measurement.
	Device can not be powered on to ready stage.	Replace with a new battery.

Specifications

Specification	HT801
MEASUREMENT TEMPERATURE RANGE	34.0~42.2°C
	(93.2~108.0°F)
AMBIENT RANGE	10~40°C
	(50~104°F)
STORAGE RANGE	-20~50°C
	(-4~122°F)
MEASURE TIME	About 1 sec
TIME BETWEEN MEASUREMENT	< 5 sec
DISPLAY MODE	ORAL
SCALE SELECTION	°C / °F
DISPLAY TYPE	LCD
MEMORY	9 set
DISPOSABLE PROBE COVER	Without Probe Cover
ACTIVATION	SCAN BUTTON
DESIGN	HANDHELD
SENSOR TYPE	Thermopile
BATTERY TYPE	CR2032 * 1pc

WARRANTY

Manufacturer's 2 Year Limited Warranty

Use of this product, whether arising out of any breach of this warranty or any legal theory, all expressed and implied warranties, including the warranties of the merchantability are limited to the applicable warranty period set forth above. Manufacturer and Distributor assume no responsibility for injuries, damages or penalties incurred or suffered resulting from the use of this product in an improper manner or location other than that for which it is intended or from failure to heed any warning disclaimer or instruction. Some states do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above exclusions or limitations may not apply to you. This warranty gives your specific legal rights and you may also have other rights which vary from state to state. Should your product become defective during this warranty period, please return the product in the original carton prepaid to:

Hyundai Healthcare USA
3250 Wilshire Blvd
Suite 813
Los Angeles, CA 90010

www.hyundaihealthcare.com

Include \$7.95 for return shipping and insurance (non-refundable)

Include with your monitor a letter stating:

1. Your name and street address
2. Daytime phone number
3. Detailed description of the problem
4. Copy of your dated cash receipt or other proof of purchase showing date

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