
Automatic Wrist

Blood Pressure /
Pulse Monitor

Model: HW201

HYUNDAI

Instruction Manual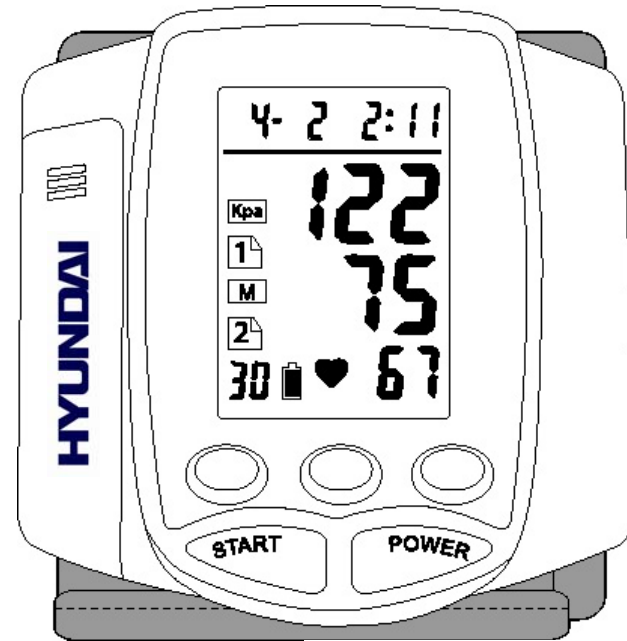

CE FDA
0434 KO14141

Table of Contents

Page

Battery Installation	1
How to Set and Date	2
LCD Panel Display	3
Before Taking your Measurements	4
How to Measure your Blood Pressure	5 - 7
Blood Pressure Classification	8
Information on Blood Pressure	9
Why Monitor at Home	10
Memory	11
Taking Care of your Monitor	12
Troubleshooting	13
Warranty	14
Specifications	15

Hyundai Preventative Care

Thank you for purchasing Hyundai's Automatic Wrist Blood Pressure Monitor!

Purchasing this product indicates that you are now on your path to a healthier lifestyle through preventative care. We at Hyundai are dedicated in providing quality products so those that practice preventative care can live a longer, healthier life.

Monitoring your blood pressure at home is your first step to a healthier lifestyle but we at Hyundai recommend you consult your doctor regularly, stay active through exercise and eat healthy. Adjusting your diet to include more fruits and vegetables, whole grains, fish and low/non-fat dairy products is a great way to start. Also make sure to check on your sodium intake. As always, consult your doctor before beginning any dietary changes.

Important Warnings/Medical Disclaimer


Prior to using your blood pressure monitor, you should read, understand and follow all instructions and warnings.

Information in this manual is provided for information purposes only. This manual and product are not meant to be a substitute for the advice provided by your physician or any other medical professional. You should not use the information contained herein or this product for diagnosing or treating a health problem, disease or prescribing any medication. Should you suspect or have a medical problem, promptly contact your personal physician, healthcare provider or call 911.


Hyundai blood pressure monitors are recommended for use by people over the age of 18 and are not suitable for clinical use.

BATTERY INSTALLATION


1. To insert the batteries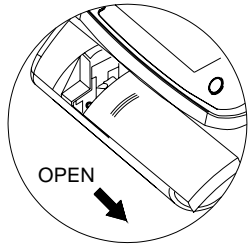

1. Remove the battery cover in accordance with the direction of arrow.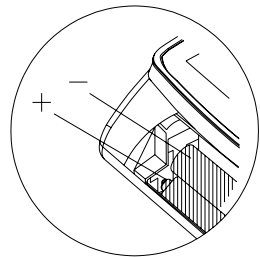

2. Insert 2 x "AAA" battery in accordance with the indication of polarities on the batteries.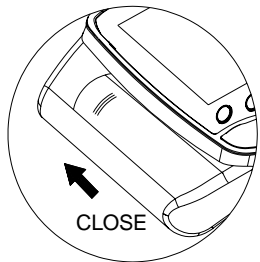


3. Slide the battery cover in accordance with the direction of the arrow to close.

HOW TO SET TIME AND DATE

Generally, LCD panel displays Month / Date / Hour / Minute Regarding time message. If you desire to adjust Year / Month / Date and Time setting, please follow The procedures as below:

1. Press down "TIME" button at least 5 seconds.
2. You can start "year" setting when "year" digit blinks.
3. Then you press "MEMO" to forward "Year" Setting.
4. You also can press "PAGE" to backward "Year" Setting.
5. After finishing Year Setting, you can press "TIME" button again.
Now, you can adjust Month / Date / Hour and Minute Setting when Month /Date/Hour and Minute digits blink as you desire.

LCD PANEL DISPLAY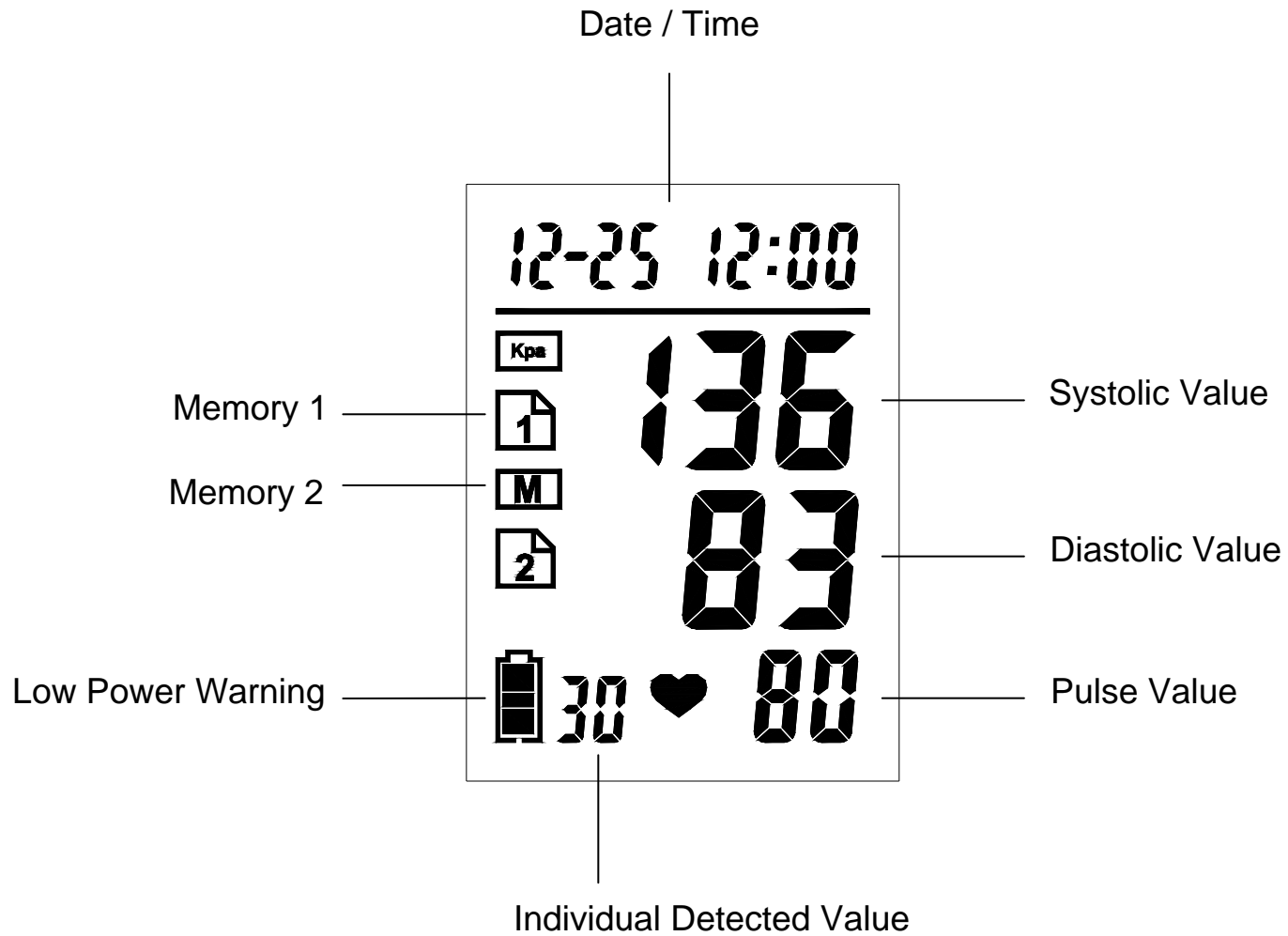

Before taking a measurement:

1. Sit quietly for about 10-15 minutes, this will allow your body to return to its normal, resting state.
2. Take your blood pressure in a comfortable environment.
3. If you have consumed any beverages containing caffeine such as coffee, teas, or cola, wait about 30-45 minutes. Do not smoke prior or during taking a measurement.

While taking a measurement:

1. Sit quietly and still while measuring. Talking or moving may lead to elevated measurements.
2. For consistency, it is suggested using the same wrist and at about the same time each day. We suggest using your left wrist if possible.
3. When taking multiple measurements right after each other, wait at least 10 minutes in between. This allows your blood pressure to return to their normal state.
4. Follow the instructions in this manual on how to measure your blood pressure.


Note: Blood pressure constantly changes during the course of a day. Occasionally, high and low readings do not necessarily indicate your monitor is malfunctioning or that a serious condition exists. It is important to speak with your doctor to learn what your normal blood pressure levels should be at and if you should experience related abnormal measurements. Should you feel you cannot wait to speak with your doctor, call 911.

HOW TO MEASURE BLOOD PRESSURE


HOW TO MEASURE BLOOD PRESSURE

1. To start the measurement

Note: In order to get correct measurement and avoid unnecessary erroneous measurement, user should not talk, be seated upright and fasten the cuff securely around bare skin as shown in pictures below.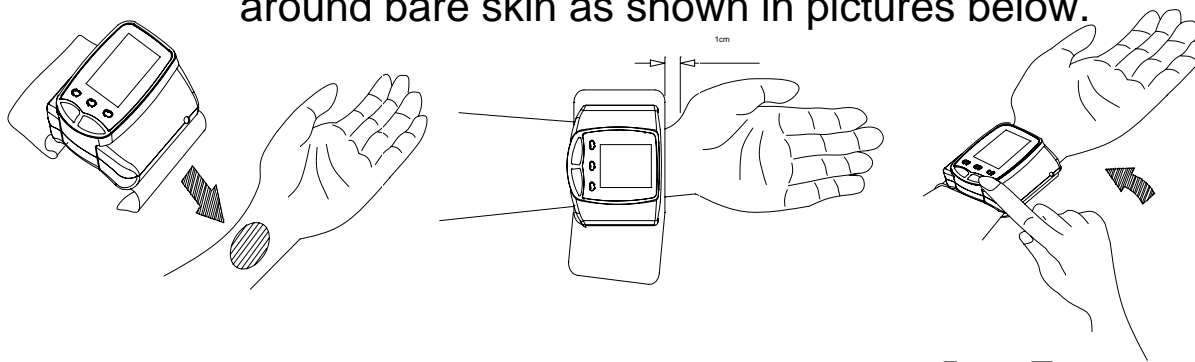

1. Place your elbow on table and turn your palm up. Make sure the cuff is at the same level as your heart. If you cannot keep your arm completely still throughout the reading, your storage case may be used to support your arm.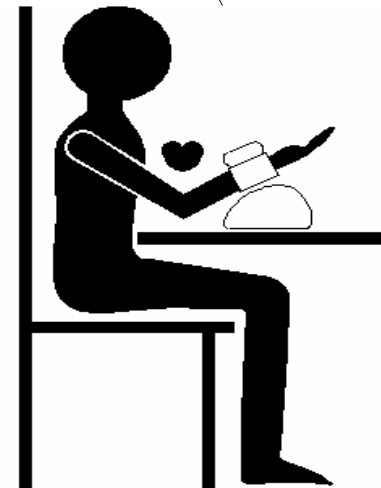

2. Press the "POWER" button.
3. Press the "START" button after blinking digits stop.
4. Start measuring, the cuff will inflate automatically.
5. After detecting Systolic / Diastolic and Pulse Values, the measured values will be displayed on LCD panel screen.
6. To measure 2nd person, after pressing Power and blinking digit stops, press Page, the number 2 will appear, press Start

2. To view the previous measured values (Memory)

The main unit automatically stores the previous measured values.

(Even after batteries are replaced, the previous values are stored in 1 minute.)

1. Power off and then power on your unit.
2. Press "MEMO" button, this will enable you to recall previous values for the first person.
3. If you press "MEMO" button successively, then you can read next values.
4. Press "PAGE" button, then you can read second person's measured values.
5. Press "MEMO" button successively to continue checking second person's values.
6. Stored capacity is up to 30 sets each page (60 total).

3. To finish the measurement


1. After deflation, the measurement of blood pressures is completed.
If you want to measure again, just press “START” button again.
2. Press “POWER” button to switch off electrical supply.
3. If you forget to push the “POWER” button, the power will turn off automatically within 3 minutes.

LOW POWER WARNING

Normally, you can measure your pressure values up to 150~200 times with durable alkaline batteries. If battery icon shows “low power warning” or show only 1 bar, then it’s time to change batteries.

BLOOD PRESSURE CLASSIFICATION

Standards for assessment of high blood pressure, without regard to age, have been established by the World Health Organization (WHO), as shown in the chart as below :

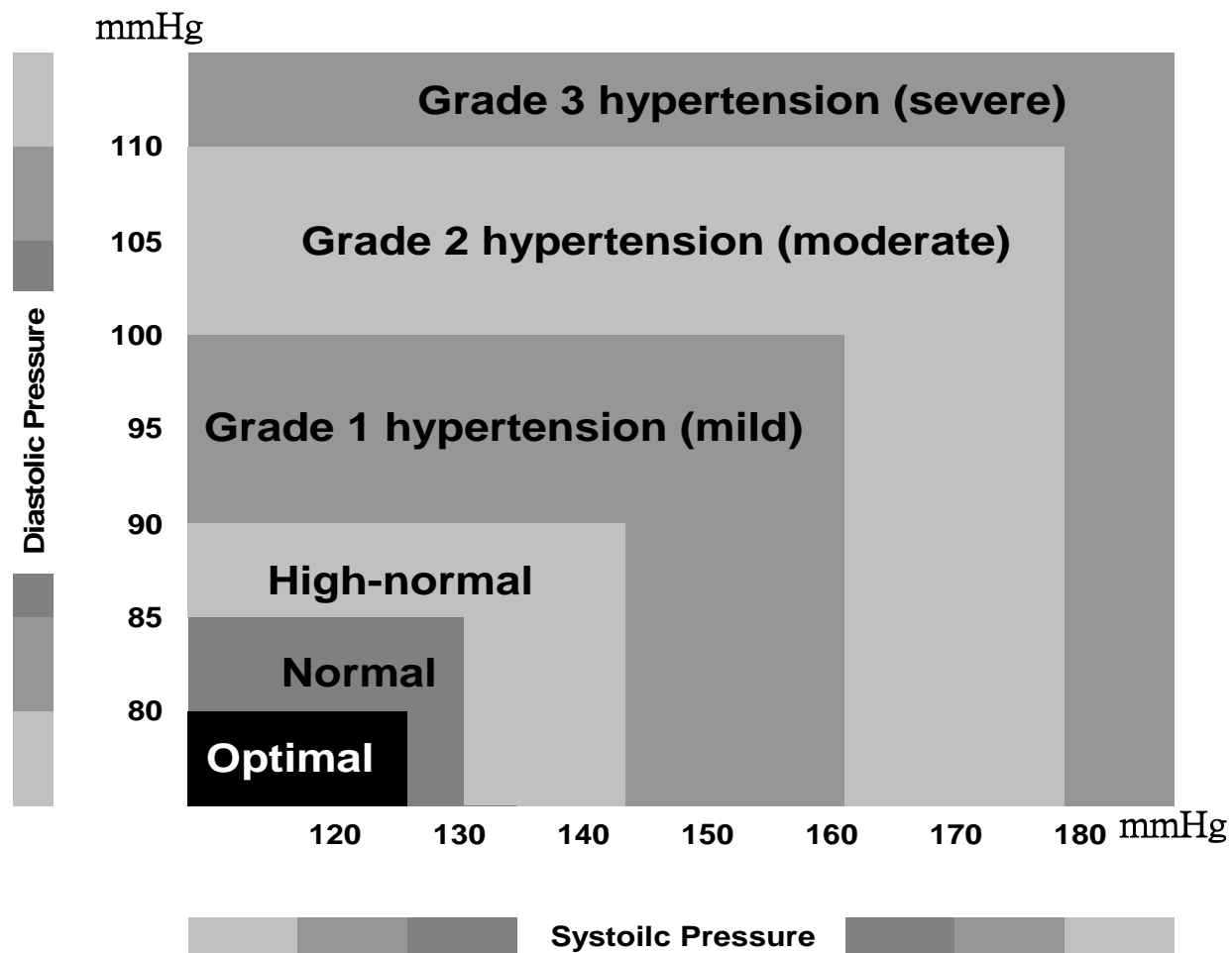INFORMATION ON BLOOD PRESSURE

What is blood pressure?

Blood is sent to the arteries with the action of a pump called heart (contraction and dilation). The pressure of blood leaving the heart is called “blood pressure”.

Blood pressures pulsate with each beat of the heart. The high blood pressure when the heart contracts are called “systolic pressure” and the low blood pressure when the heart dilates is called “diastolic pressure”. The threshold value for hypertension in adults is defined by the World Health Organization (WHO) as 140/90 mmHg.

Health and blood pressure

When people reach middle age, the risk of hypertension markedly increase.

With aging, the blood vessels age rapidly. Furthermore, because of the obesity and lack of exercise, cholesterols stick to blood vessels, causing them to lose elasticity. Therefore, watching daily blood pressure help to evaluate our health condition.

Why do we need to monitor blood pressure at home?

By monitoring at home, you may manage your health between doctor visits. Home monitoring also provides you with a record of blood pressure measurements over time to give your doctor - a valuable tool in hypertensive monitoring. Home monitoring may also alleviate white coat hypertension. This term refers to the phenomena which causes individuals' blood pressure to be higher when taken in a medical office setting. This is due to the stress of being in a medical office setting which may cause blood pressure to rise higher than normal. When monitoring at home, you may get a more accurate measurement of your blood pressure in a normal, relaxed state.

Sixty Measurement Memory

When you take a measurement, this unit will automatically store in memory. Your unit is capable of storing up to 60 measurements, 30 for each page. To review your measurements and how to store, follow the instructions in the manual.

Taking Care of Your Blood Pressure Monitor

- Keep the unit out of direct sunlight, high humidity and extreme temperature changes.
- Do not drop on the floor
- Do not take the unit or the cuff apart. Should your unit require repairs, see details in warranty section.
- Keep the unit free of dust and other materials. A storage container is provided for safe storage. Avoid getting any parts of the units or cuff wet. Should the unit need cleaning, it is best to wipe it with a cloth slightly dampened with water. Hyundai recommends cleaning your unit once a month.
- Do not fold or bend the cuff with force, this may damage the unit.
- Do not inflate the monitor's cuff if it is not wrapped around your wrist.
- For your safety, do not wrapped the cuff around any other body part other than your wrist.

TROUBLES SHOOTING

Symptoms	Problem	Solution
Nothing displayed when you push the Start button.	Are the batteries exhausted?	Replace the batteries with new ones.
	Are polarities of the batteries in wrong position?	Insert batteries in the correct direction.
“Err P” is shown on LCD panel	Fail to inflate	Please check if the cuff is too loosen, and fasten it.
“Err 1” is shown on LCD panel	Deflate too fast	Please contact authorized Hyundai dealer.
“Err 2” is shown on LCD panel	<ol style="list-style-type: none"> 1. Movements occur on wrist 2. Other interference signal around BPM 	<ol style="list-style-type: none"> 1. Keep still after you push the Start button. 2. Please remove interfering sources such as magnetic material or cell phone.
“Err 3” is shown on LCD panel	Incorrect measured result	<ol style="list-style-type: none"> 1. The cuff might be in incorrect position. Please re-wrap the cuff in correct position. 2. You might use it incorrectly, such as measure pressure value on child.
<ul style="list-style-type: none"> • If correct measurement is impossible even after checking the above-mentioned points, consult at the store where you have purchased the unit or the nearest Hyundai dealer without touching the internal mechanism. • In some very rare cases, there may be error due to the physical condition of the person. In such cases, please consult your doctor. 		

Manufacturer's 5 Year Limited Warranty

Use of this product, whether arising out of any breach of this warranty or any legal theory, all expressed and implied warranties, including the warranties of the merchantability are limited to the applicable warranty period set forth above. Manufacturer and Distributor assume no responsibility for injuries, damages or penalties incurred or suffered resulting from the use of this product in an improper manner or location other than that for which it is intended or from failure to heed any warning disclaimer or instruction. Some states do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above exclusions or limitations may not apply to you. This warranty gives your specific legal rights and you may also have other rights which vary from state to state. Should your product become defective during this warranty period, please return the product in the original carton prepaid to:

Hyundai Healthcare USA
3250 Wilshire Blvd
Suite 813
Los Angeles, CA 90010
WWW.hyundaihealthcare.com

Include \$10.95 for return shipping and insurance (non-refundable)

Include with your monitor a letter stating:

1. Your name and street address
2. Daytime phone number
3. Detailed description of the problem
4. Copy of your dated cash receipt or other proof of purchase showing date

***Note: Product maybe replaced or repaired at our option that is defective in material or workmanship**

SPECIFICATIONS

Name	Wrist Blood Pressure Monitor
Model	HW201
Measurement	Oscillometric
Measurement Range	Pressure: 20-280mmHg Pulse rate: 40-195beats/ min
Accuracy	Pressure: within ± 3 mmHg Pulse rate: within $\pm 5\%$ of reading (Subject to the temp. humidity listed below)
Inflation	Automatic inflation with pump
Pressure Detection	Semi –Conductor
Power Supply	2X "AAA" alkaline batteries
Auto Power Off	3 minutes after last key operation
Operating Temperature/ Humidity	+10°C to +40°C, 30% to 85% RH
Storage Temperature Humidity	-20°C to +60°C, 10% to 95% RH
Weight of Main Unit	Approximately: 0.31 lbs. / 140g (excluding batteries)
External Dimensions	3.15(W) x 3.35(H) x 2.36(D) inches / 80(W)X85(H)X60(D) mm
Wrist Circumference	5.31 – 8.66 inches / 135mm-220mm

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