



OPERATING INSTRUCTIONS

MDF 767 Sprague Rappaport Stethoscope (MDF 767B and MDF 767K)
MDF 767X 2-N-1 Sprague Rappaport Stethoscope

Read instructions carefully before using and save for future reference.

General Safety: MDF stethoscope is exclusively designed as a non-invasive medical auscultation instrument to detect heart and lung sounds on healthy skin around the external thoracic region. Due to its sound amplification function, it may cause damage to user's ears if used other than its intended medical diagnostic purpose. Use only if you are a medical professional and do not leave stethoscope unattended around children.

General Usage: MDF stethoscope headsets are angled anatomically for best fit, comfort and sound transmission. The headset should be worn with the eartips pointed away from you if it is held directly in front of you.

The Sprague Rappaport chestpiece and attachments offers 5 different auscultation functions:

- Adult Diaphragm Attachment** – Auscultation of high frequency sounds on adult patients.
- Pediatric Diaphragm Attachment** – Auscultation of high frequency sounds on pediatric patients.
- Adult Bell Attachment** – Auscultation of low frequency sounds on adult patients.
- Pediatric Bell Attachment** - Auscultation of low frequency sounds on pediatric patients.
- Infant Bell Attachment** - Auscultation of low frequency sounds of infant patients.

Select attachment, screw clockwise and turn the acoustic valve stem to desired side of the chestpiece. Hold the chestpiece firmly against the desired location on the patient's external thoracic region. Prior to each use, make sure all parts are securely fastened and properly connected.

General Cleaning & Care: Clean stethoscope thoroughly before each use. For chestpiece, acoustic tubing and ComfortSeal® eartips, use clean cotton cloth or swab and rubbing alcohol or isopropyl alcohol solution to cleanse all soiled parts. Avoid prolong exposure of the acoustic tubing to excessive heat, cold, solvents, moisture and oils/lipids (secreted by human skin).

Our commitment to high-quality instruments is backed by our two-year performance guarantee.



MDF®, and ComfortSeal® is a registered trademark by MDF Instruments USA, Inc.
© 2006 MDF Instruments USA, Inc. All Rights Reserved.



- MDF Instruments Direct, Inc. 28947 Thousand Oaks Blvd. Suite 222 Agoura Hills CA 91301 USA
- MDF Instruments Europe ApS. Kronprinsessegade 44, kld. 1306 Copenhagen K Denmark
- MDF Instruments Canada, Inc. 566A Concession St. Hamilton Ontario L8V 1B1 Canada
- MDF Instruments USA, Inc. 26665 Seagull Way Suite 117 Malibu CA 90265 USA
- MDF Instruments Germany Lindenstraße 14 D-50674 - Köln Germany
- MDF Instruments Asia 1539 Wu Zhong Rd. Shanghai 201103 China